



GE Healthcare

GE Healthcare, Surgery  
384 Wright Brothers Drive  
Salt Lake City, Utah 84116

## CUSTOMER PRODUCT IMPROVEMENT

April 16, 2008

**To:** OEC Trained/Qualified Radiological Technologists  
Director/Manager of Radiology

**Subject:** Vertical Lift Column Power Supply (PS3) Replacement

**Affected Products:** OEC® 8800; OEC® 9800; OEC® 9900 Elite

The Vertical Lift Column Power Supply (PS3) has been replaced on this GE Healthcare OEC C-arm. The replacement PS3 eliminates the early life failure of the previous version.

The new PS3 functions slightly different than the older version. The Up and Down switches for the Vertical Lift Column operate as before and in order to protect the power supply, a delay (0.4 second) was built into the Up command and Down command switch operations. If the user makes a quick change between Up and Down commands, or presses the same button quickly in succession, the respective switch will need to be released and pressed again. This delay is normal and expected, and will protect the PS3 from failure.

The overall benefit to the new Vertical Lift Column Power Supply is the elimination of a potential Vertical Lift Column failure that has been identified as a safety concern. The built in switch delay makes the integrity of the PS3 consistent with the overall life expectancy of your GE Healthcare OEC C-arm, and will eliminate the failure of the Vertical Lift Column during use.

**If you have any questions or concerns regarding this issue, please do not hesitate to contact the sales or service team for further information at 800-874-7378. Information is available at this number 24 hours per day, 7 days a week.**